

A1500-S2-V120 Reciprocating Medical Air System

System is NFPA compliant and has modular construction and interconnecting piping.

PUMP MODULE

The belt driven oil-less piston compressor delivers pressures up to 115 psi. Each module includes the compressor; motor; drive belts with belt guard; inlet filter; isolation valve; aftercooler; temperature switch for each cylinder, unloader solenoid, and relief valve.

AIR TREATMENT MODULE

The system's air treatment module features dual desiccant dryers configured for energy savings. They purge only when needed per the dew point setting. The dew point and CO transducers are integrated with the electrical panel. The module also contains prefilters, after filters, and activated carbon filters. Final line pressure regulators with gauges and relief valves are also part of the air treatment module.

AIR RECEIVER

The receiver is constructed to ASME standards and is galvanized for corrosion resistance. A source isolation valve and 3 valve bypasses are provided with the receiver module. A pressure gauge and relief valve are included, along with a sight window and electric auto drain. Mounting pads and flexible connectors are provided.

CONTROL PANEL

The UL listed electrical control panel features a high-definition color touch screen that can be set up externally without the use of toggle switches. The panel displays multiple conditions, including compressor running status, elapsed hours, pressure level, and audio/visual alarms.

Standard features include lag alarm, automatic compressor alternation, adjustable minimum run timer, adjustable pressure settings, adjustable screen brightness, alarm history registry, and data logging. The panel has standard dry contacts and Modbus interface with building management systems. Ethernet and BACnet are available as options. The panel's maintenance menu includes service items and maintenance warnings, and it is programmed for both English and Spanish languages.

Configuration	Duplex Stack Mount
System Capacity*	60 SCFM @ 50 psi 60 Hz. 50 SCFM @ 50 psi 50 Hz.
Horsepower (Each Pump)	15 HP
Horsepower (Total System)	30 HP
Available Voltage	208/230/460 v. 3 phase 60 Hz. 380 v. 3 phase 50 Hz. Contact Factory for other voltages
Full Load Amps 3 ph.	208 volts: 96.6 230 volts: 87.8 380 volts: 51.2 460 volts: 44
Tank Size	120 gallon vertical
Key Options	Bacterial filters for dryer/filter skid. Can be configured for lab use without lag alarm. Expandable for additional compressors.


Click here for compressor specifications and industry-leading parts support.




*System capacity is stated with one pump in reserve per NFPA. Capacity based on operation at sea level. For higher altitude, contact factory for adjusted capacity.

SYSTEM MODEL NO.	PART NO.	HP EACH	SCFM/ACFM @ 90 PSIG EACH	SCFM/ACFM @ 100 PSIG EACH	DESIGN CAPACITY @ 300 PSIG EACH	RECEIVER SIZE (GALLONS)	PUMP DISCHARGE NPT	ESTIMATED SYSTEM WEIGHT (LB)
A1300S-52-DD	ADM1300(C)-440-52-V120-DD	15	59	55	56	120	1"	2400

SPACE ALLOWANCE FOR MAINTENANCE = 3 FT. ON ALL SIDES. MORE SPACE IS REQUIRED FOR THE OPENING OF LARGER PANEL DOORS.

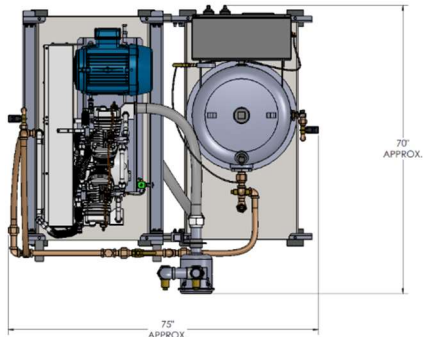

AIR COMPRESSOR MODULE

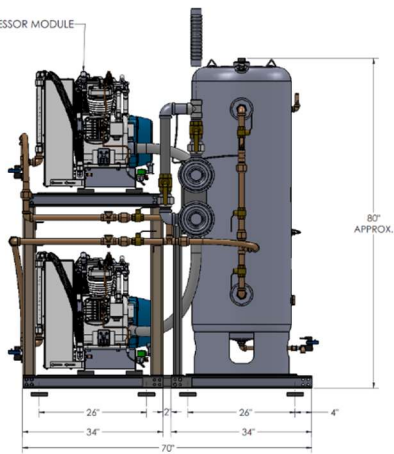


2" NPT AIR INLET CONNECTION

AIR INTAKE FILTERS

1" NPT CONNECTION TO DRYER'S INLET

AIR RECEIVER

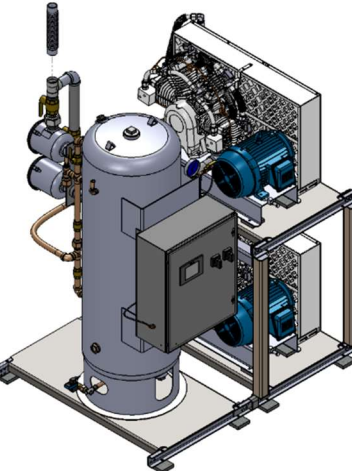

ELECTRICAL CONTROL PANEL

TABLE 1

SYSTEM LISTED ARE 480V. IF ANOTHER VOLTAGE IS REQUIRED, USE THE FOLLOWING PART NUMBERS.

VOLTAGE	SYSTEM P/N	PUMP MODULE P/N	TRP MODULE P/N
208-480-3	ADM1300(C)-208-52-V120-DD	100-0224	214339
230-480-3	ADM1300(C)-230-52-V120-DD	100-0224	214339
380-50-3	ADM1300(C)-380-52-V120-DD	100-0224	214544

REV	DATE	DESCRIPTION	BY	CHKD
01	06/01/2021	INITIAL RELEASE		
02	06/01/2021	REWORK		
03	06/01/2021	REWORK		
04	06/01/2021	REWORK		
05	06/01/2021	REWORK		
06	06/01/2021	REWORK		
07	06/01/2021	REWORK		
08	06/01/2021	REWORK		
09	06/01/2021	REWORK		
10	06/01/2021	REWORK		
11	06/01/2021	REWORK		
12	06/01/2021	REWORK		
13	06/01/2021	REWORK		
14	06/01/2021	REWORK		
15	06/01/2021	REWORK		
16	06/01/2021	REWORK		
17	06/01/2021	REWORK		
18	06/01/2021	REWORK		
19	06/01/2021	REWORK		
20	06/01/2021	REWORK		
21	06/01/2021	REWORK		
22	06/01/2021	REWORK		
23	06/01/2021	REWORK		
24	06/01/2021	REWORK		
25	06/01/2021	REWORK		
26	06/01/2021	REWORK		
27	06/01/2021	REWORK		
28	06/01/2021	REWORK		
29	06/01/2021	REWORK		
30	06/01/2021	REWORK		
31	06/01/2021	REWORK		
32	06/01/2021	REWORK		
33	06/01/2021	REWORK		
34	06/01/2021	REWORK		
35	06/01/2021	REWORK		
36	06/01/2021	REWORK		
37	06/01/2021	REWORK		
38	06/01/2021	REWORK		
39	06/01/2021	REWORK		
40	06/01/2021	REWORK		
41	06/01/2021	REWORK		
42	06/01/2021	REWORK		
43	06/01/2021	REWORK		
44	06/01/2021	REWORK		
45	06/01/2021	REWORK		
46	06/01/2021	REWORK		
47	06/01/2021	REWORK		
48	06/01/2021	REWORK		
49	06/01/2021	REWORK		
50	06/01/2021	REWORK		
51	06/01/2021	REWORK		
52	06/01/2021	REWORK		
53	06/01/2021	REWORK		
54	06/01/2021	REWORK		
55	06/01/2021	REWORK		
56	06/01/2021	REWORK		
57	06/01/2021	REWORK		
58	06/01/2021	REWORK		
59	06/01/2021	REWORK		
60	06/01/2021	REWORK		
61	06/01/2021	REWORK		
62	06/01/2021	REWORK		
63	06/01/2021	REWORK		
64	06/01/2021	REWORK		
65	06/01/2021	REWORK		
66	06/01/2021	REWORK		
67	06/01/2021	REWORK		
68	06/01/2021	REWORK		
69	06/01/2021	REWORK		
70	06/01/2021	REWORK		
71	06/01/2021	REWORK		
72	06/01/2021	REWORK		
73	06/01/2021	REWORK		
74	06/01/2021	REWORK		
75	06/01/2021	REWORK		
76	06/01/2021	REWORK		
77	06/01/2021	REWORK		
78	06/01/2021	REWORK		
79	06/01/2021	REWORK		
80	06/01/2021	REWORK		
81	06/01/2021	REWORK		
82	06/01/2021	REWORK		
83	06/01/2021	REWORK		
84	06/01/2021	REWORK		
85	06/01/2021	REWORK		
86	06/01/2021	REWORK		
87	06/01/2021	REWORK		
88	06/01/2021	REWORK		
89	06/01/2021	REWORK		
90	06/01/2021	REWORK		
91	06/01/2021	REWORK		
92	06/01/2021	REWORK		
93	06/01/2021	REWORK		
94	06/01/2021	REWORK		
95	06/01/2021	REWORK		
96	06/01/2021	REWORK		
97	06/01/2021	REWORK		
98	06/01/2021	REWORK		
99	06/01/2021	REWORK		
100	06/01/2021	REWORK		

